


**PLEASE READ THIS SUPPLEMENT IN CONJUNCTION WITH THE PRO-CARE OR PRO-COMM USER MANUAL - IT COVERS PROCEDURES NOT DETAILED IN EITHER MANUAL.**

Any bed ordered as a 220cm variant will generally be supplied with the extension parts pre-fitted. If such a bed is being shipped to you via a 3rd party carrier then the 220cm long siderails will be shipped separately to the bed - so you will receive 2 consignments.


**PLEASE REFER TO THE FOLLOWING INSTRUCTIONS IF YOU WISH TO EXTEND AN EXISTING BED AND HAVE THEREFORE ORDERED AN EXTENSION KIT ONLY.**

The parts shown below are used to extend beds from 3 ranges. Refer to the table to identify the parts applicable to a particular model of bed.

**PRO-COMM MODEL BEDS (incl Pro-Bario)**

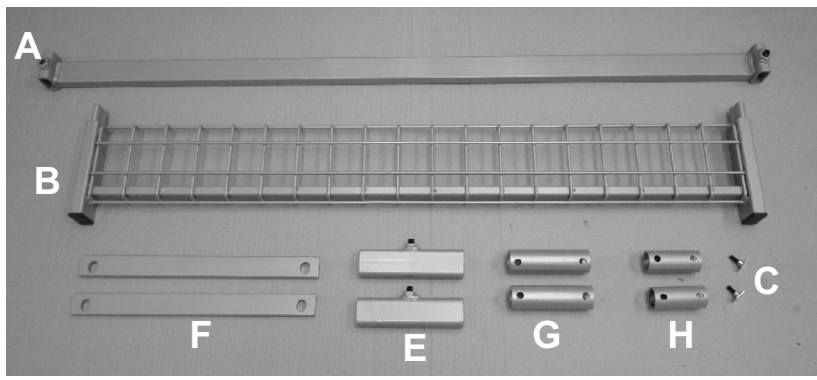

**PRO-CARE MODEL BEDS**

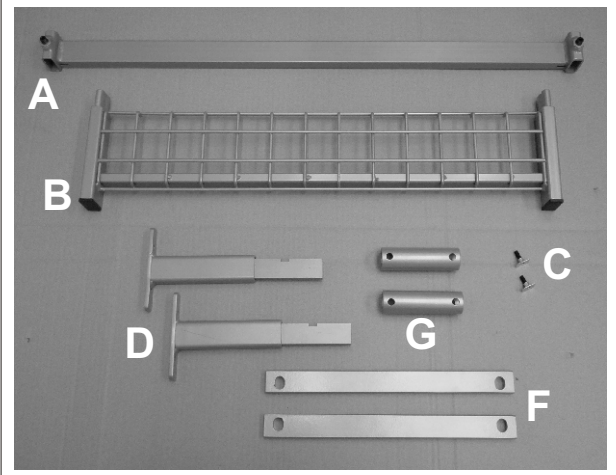

Component:	Applicable to:	Pro-Care 90cm LA31 (grey) actuators	Pro-Care 120cm LA31 (grey) actuators	Pro-Comm 90cm LA27 (black) actuators	Pro-Comm 120cm LA31 (grey) actuators	Pro-Bario 120cm LA31 (grey) actuators
<b>A</b> Mid-section 5cm extension bar		Yes	Yes	Yes	Yes	Yes
<b>B</b> Leg-lift 15cm mesh extension		Yes	Yes	Yes	Yes	Yes
<b>C</b> Fixing plate/screw for 'B' above		Yes	Yes	Yes	Yes	Yes
<b>D</b> Extended Head/Foot board 'T' bkts		Yes	Yes	No	No	No
<b>E</b> Collar - Foot end lifting assembly		No	No	Yes	Yes	Yes
<b>F</b> Extended matt. support joining bar		No	Yes	No	Yes	Yes
<b>G</b> Actuator extension sleeve - 90mmØ		No	Yes	No	Yes	Yes
<b>H</b> Actuator extension sleeve - 50mmØ		No	No	Yes	No	No


**Location of actuator extension sleeves**

On certain beds sleeves with 50mm hole centres (H) are fitted as standard - these will require replacing with the extended sleeves supplied in the kit - item (G)



**MATTRESS SUPPORT ASSEMBLY - FOOT END (120 cm wide beds only)**  
 (Codes CB2BL-200-120; CB2L-200-120; CB1-200-120)


The extended joining bar (Item F above) should be fitted to the foot end of the mattress support unit. There is a pair of grub screws at each side - unscrew sufficiently to remove the standard joining bar. Slide the extended bars into the tubular section and loosely tighten the grub screws - slide the two halves of the mattress support unit together before fitting the clevis pin and fully & securely tightening the grub screws.

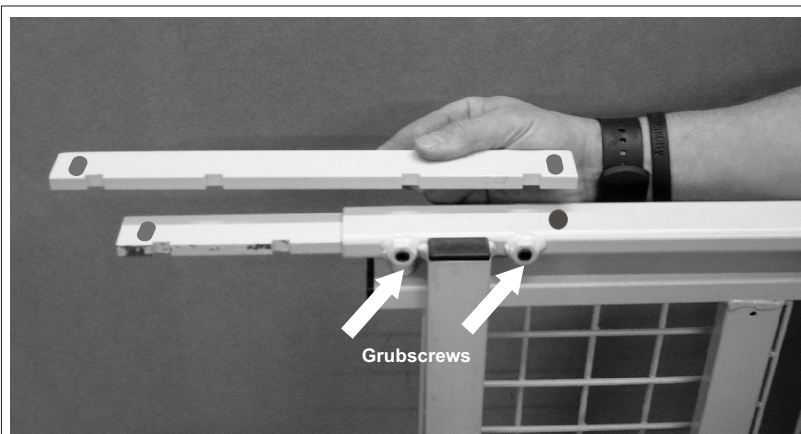

**ASSEMBLY OF MID-SECTION EXTENSION BAR.**

**ALL beds**

Slide the mid-section 5cm extension bar (Item A above) fully over the two 'tails' / joining bars on the foot end of the mattress support unit until the two sections of tubular steel touch. The grubscrews should face downwards. Securely tighten the two grubscrews.

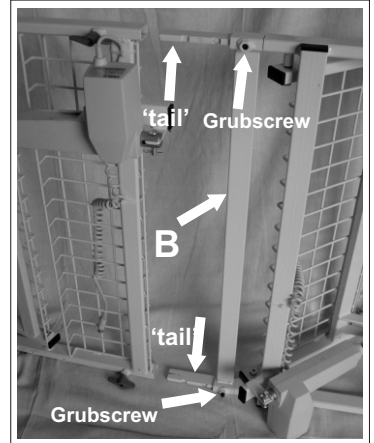

**ASSEMBLY OF LEG LIFT MESH EXTENSION.**

**ALL beds**






Remove the 2 black plastic end caps from the foot-lift assembly.

Using long-nosed pliers locate the fixing screws (Item C, page 1) from inside the tubular section through their locating holes.

Once both fixings are securely in place then position the two 'male' joining pieces of the 15cm mesh extension (Item B, page 1) into the open tubular sections - ensuring that the mesh faces the upper side of the bed.

The extension is designed to be tight fitting and may require tapping into place with a rubber hammer.

Ensure that it is fully inserted until the outer tubes touch. Securely tighten the two grubscrews.

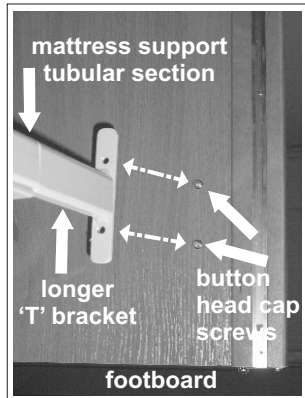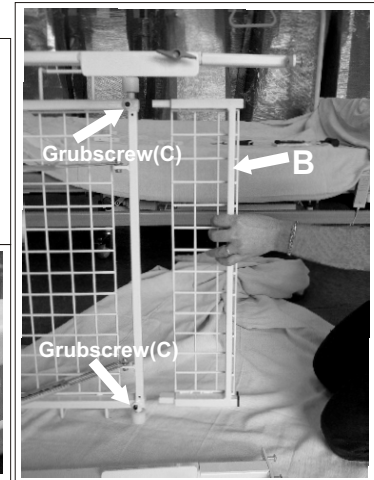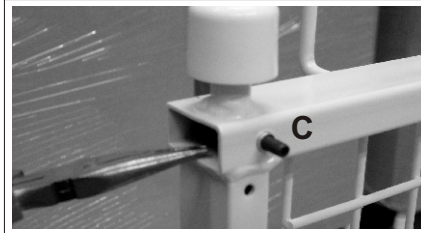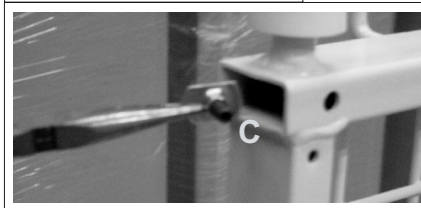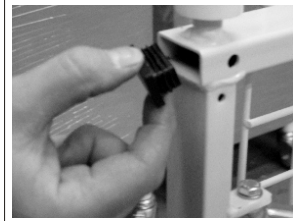


**FOOT BOARD**

**PRO-CARE beds**

The 2 standard length 'T' brackets on the footboard should be removed by carefully unscrewing the 4 button-head cap screws. Align each extended & collared 'T' bracket (Item D, page 1) with the mounting holes in the footboard and securely fix in place using the button-head cap screws.

**BED END LIFTING ASSEMBLY**  
**PRO-COMM & PRO-BARIO beds**

Unlike Pro-Care beds the 'T' bracket used to attach the bed end lifting assembly to the mattress support unit on Pro-Comm style beds is fixed. A collar is used to occupy the space taken by the 15cm mesh extension. Slide one of the collars over each 'T' bracket as shown and securely tighten the grubscrews.

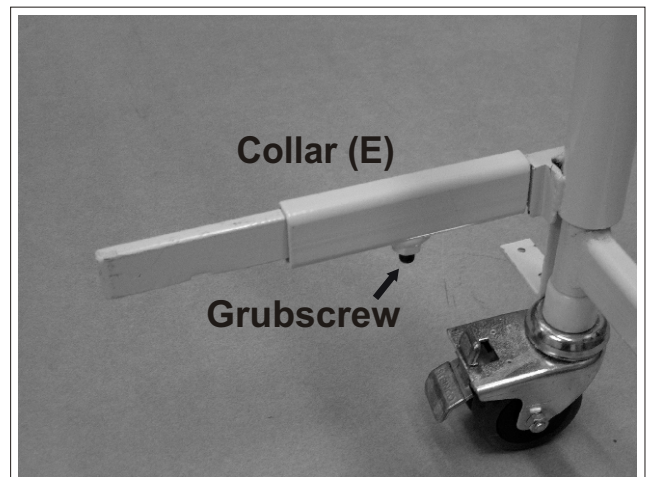

**YOU SHOULD NOW REFER TO EITHER THE PRO-CARE OR PRO-COMM USER MANUAL FOR DETAILED INSTRUCTIONS ON RE-ASSEMBLING THE BED.**