

The packaging of this mattress is sealed prior to despatch and the mattress itself will be clean prior to closure. If the mattress has become exposed or damaged during transit, please contact us at the address below.




## USER MANUAL

(please read carefully)

# CRASHEX CRASH / SAFETY MAT

*(CB9-CEX)*

Combustion modified foam mattress. Waterproof, removable cover and sewn seams.

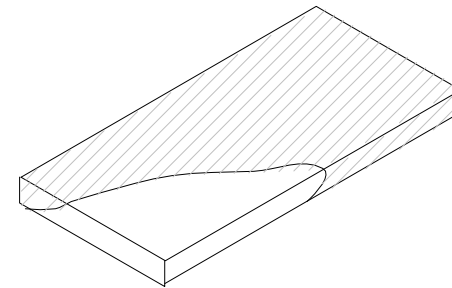

NEXUS DMS LTD

Unit 11 Lovett Road  
Hampton Lovett Industrial Estate  
Droitwich UK  
WR9 0QG

Tel +44 (0)1905 774695 Fax +44 (0)1905 796081  
Email [sales@nexusdms.co.uk](mailto:sales@nexusdms.co.uk)  
Web: [www.nexusdms.co.uk](http://www.nexusdms.co.uk)

*July 2011*

## Flammability

The Crash Mat CB9-CEX meets the requirements of BS7177: 2008 for medium hazard.

The interior materials meet the requirements of The Furniture & Furnishings (Fire) (Safety) Regulations 1988.

## User Instructions

The Crash Mat CB9-CEX is intended to be placed alongside the bed of a vulnerable patient where there is a risk of their falling. Carers should note that it will not necessarily prevent injury but will act as a cushion to any fall.

The label on this safety mat has a unique batch number which should be used in the event of a warranty claim. NEVER remove the label.

## Warranty

Covered against faulty manufacture and materials by replacement.

Normal expected wear and tear in use is not covered.

All warranties are subject to the terms and conditions of trading.

Replacement value under warranty reduces proportionally to the period in use up to 3 years after purchase.

If making a claim under warranty please quote the production identification batch number printed on the interior foam core.

## General Cleaning

- Wipe down with a suitable sponge using a solution of warm water (maximum of 40°C) and neutral detergent
- Check your chosen detergent on a small inconspicuous surface area prior to general cleaning
- **Do not** machine wash or autoclave
- **Do not** tumble dry
- **Do not** use abrasive or phenolic-based cleaners
- To disinfect in situ, use a 0.1% sodium hypochloride solution (1,000 ppm chlorine)
- Always wipe dry

Contaminated foam cores cannot be decontaminated and should be disposed of safely, as should any cover damaged by cuts, tears or 'sharps' penetration.

## Storage & Care

Store the safety mat flat – do not fold

Do not place heavy or sharp objects on the safety mat surface when in use / in store.

Do not allow sharp objects to penetrate the cover coating (incl. hypodermic needles)

Keep the safety mat clean

Regularly inspect the interior core and exterior cover of the safety mat for damage or contamination.

Do not store in damp conditions