

# INSTALLATION & USAGE

CB1 series Pro-Care beds &  
CB2 / CB21 series Pro-Comm beds.

Nexus DMS Ltd  
Unit 11, Lovett Road  
Hampton Lovett Ind. Est.  
Droitwich, Worcs, UK. WR9 0QG  
Tel: 01905 774695 Fax: 01905 796081  
  
www.nexusdms.co.uk  
sales@nexusdms.co.uk

## PACK CONTENTS


The easyEX pack contains:

- |                       |                    |
|-----------------------|--------------------|
| 1 x Sliding handle    | 1 x Mounting plate |
| 2 x Mounting brackets | 4 x 35mm screws    |
| 1 x 4mm Allen Key     |                    |




Spring-loaded swivel release pin

## POSITIONING THE easyEX MOUNTING PLATE

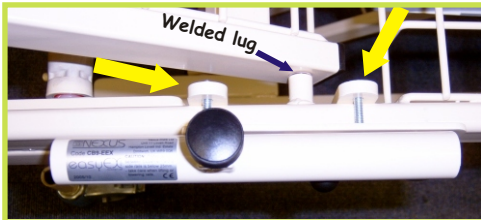

Note position of the 2 mounting brackets - either side of the welded lug.

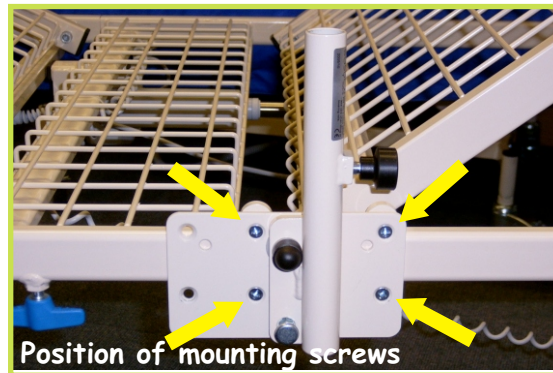


easyEX can be fitted to either side of the bed.  
Note however that the 'out of use' stowed position of the handle is affected by left or right positioning.



Position the mounting screws through the mounting plate and offer up to the mounting brackets. Screw in & tighten equally & securely using the allen key provided - ensuring that the brackets are parallel to the mounting plate when tight.  
NOTE: the

Once installed, slide the handle into the tube until it hits the spring-loaded adjustment pin. Pull the pin out and hold. Slide the handle to the chosen height & release the pin. Gently move the handle up or down until the pin locates in the nearest adjustment hole. Screw in and tighten the black knob.

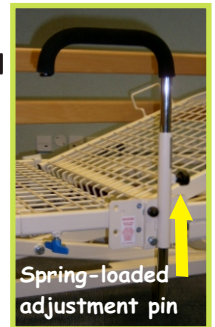


Spring-loaded adjustment pin

**IMPORTANT**  
Before using the handle ensure that the spring-loaded adjustment pin fully locates in a hole & the black knob is screwed in & tightened

**CAUTION**

1. As the gap between easyEX and timber siderails is very small caution should be exercised when raising and lowering the siderails.
2. When fitting easyEX to a bed with timber siderails, safety foam siderail bumpers can be used when the siderails are raised but must first be removed before lowering them.
3. easyEX can be fitted to Pro-Comm style beds with METAL siderails. However to use easyEX in its vertical position the siderail must first be removed. For night-time safety fold easyEX to its horizontal position and re-fit the siderail in its raised position.
4. easyEX carries a recommended safe leverage of 120kg.

## DIMENSIONS

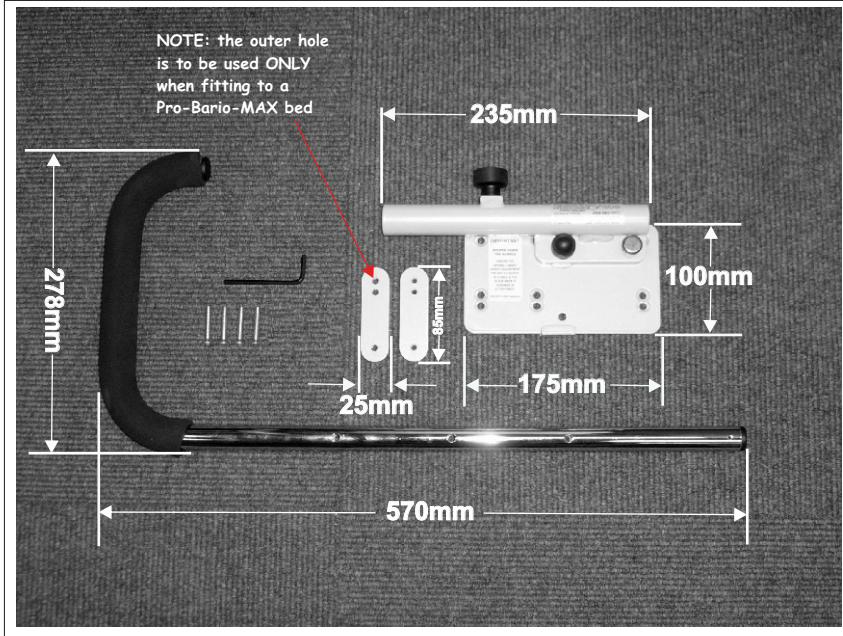

**easyEX** is manufactured specifically for beds in the Nexus range. Responsibility cannot be accepted for this product if used on other makes of bed or on Nexus beds with other makes of mattress.

