



**2-part finger, wire loop, spacer assembly instructions.
Applicable to beds up to & including Ser. No: 080998
(can be retrospectively applied to earlier beds)**

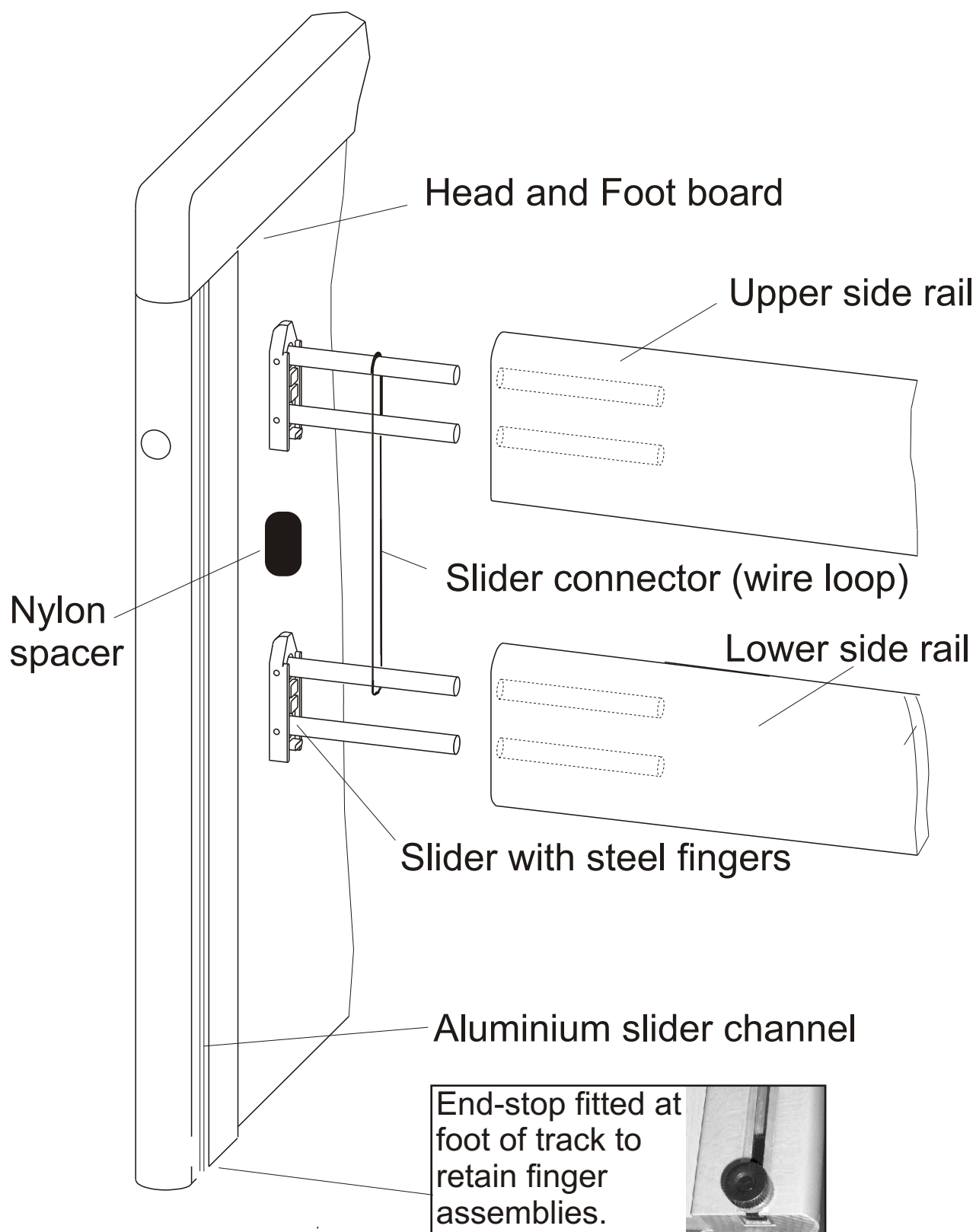

NOTE: The following procedure **must** be observed in order to maintain the correct and safe spacing between the siderails - in both raised & lowered positions:

1. The **pointed** end of each finger assembly must point **upwards**.
2. The slider connector (wire loop) must be placed **exactly as shown** around the **uppermost** finger of the top assembly & around the **uppermost** finger of the lower assembly.
3. The slider connector (wire loop) **must** be positioned **inside** the aluminium slider channel.
4. The nylon spacer **must** be inserted in the channel between the two finger assemblies.

2-part finger, wire loop, spacer assembly instructions.
Applicable to beds from Ser. No: 080999
(Can be retrospectively applied to earlier beds)

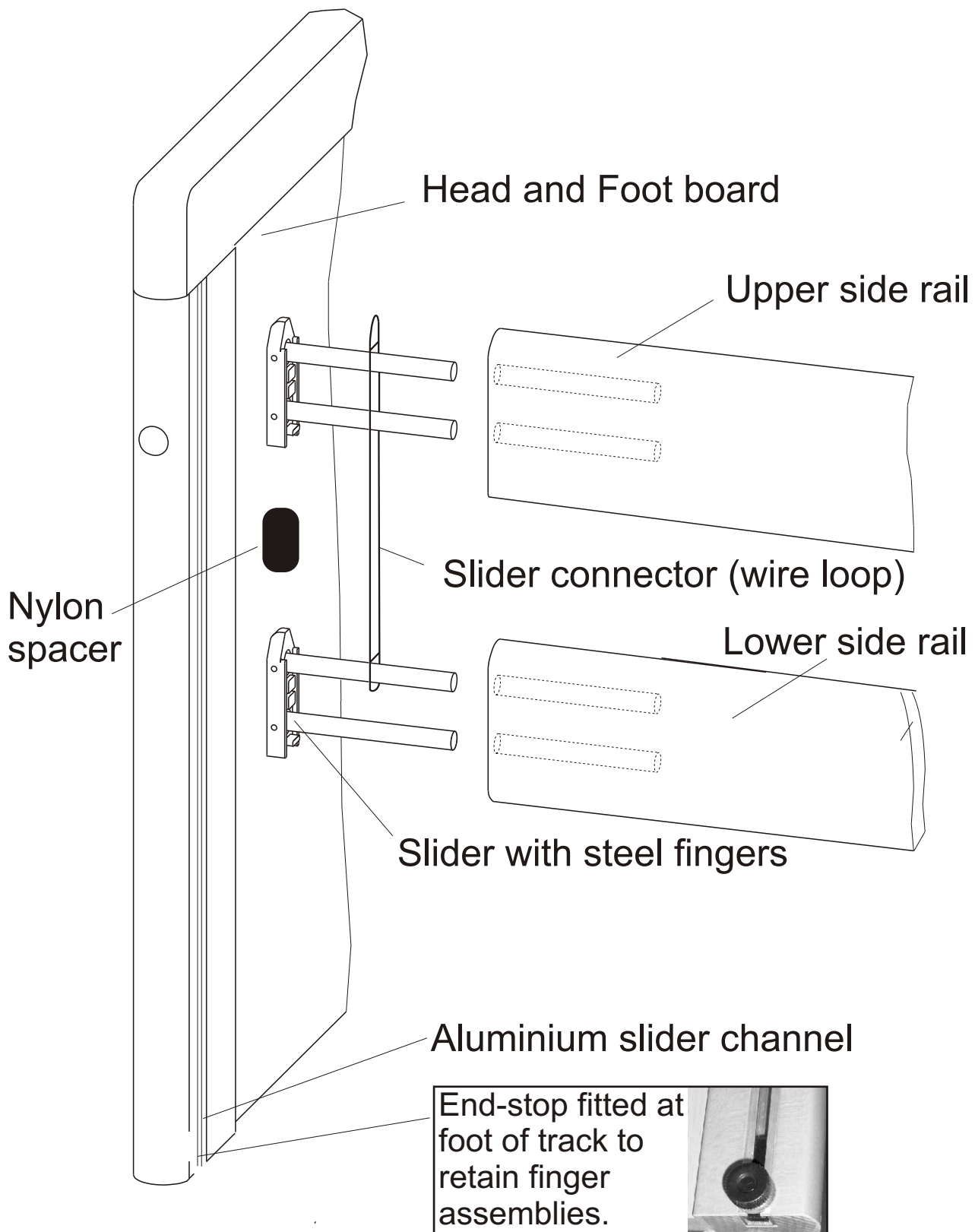

NOTE: The following procedure **must** be observed in order to maintain the correct and safe spacing between the siderails - in both raised & lowered positions.

1. The **pointed** end of each finger assembly must point **upwards**.
2. The slider connector (wire loop) must be placed **exactly as shown** around the **uppermost** finger of the top assembly & around the **uppermost** finger of the lower assembly.
3. The slider connector (wire loop) **must** be positioned **inside** the aluminium slider channel.
4. The nylon spacer **must** be inserted in the channel between the two finger assemblies.